
Securilog Update Instruction Guide




If Internet Explorer is not your usual browser, in order to provide generic instructions, we advise using it to update. If you are prompted to make Internet Explorer your default browser we suggest selecting no.

1 Go into Start - All programs - Internet Explorer

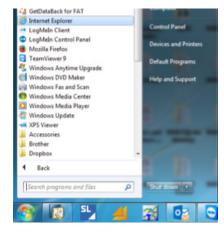

Go to the address bar at the top and type in www.Securilog.co.uk

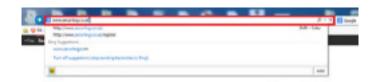

3 Click Download the latest SecuriLOG software at the top

SecuriLOG Software Download


Download the latest SecuriLOG software

It will then begin the download asking if you want to open, save or cancel. Click open and let it download




This will then take a minute to download a file and open up a box with several files inside.

Double click the file named Securilog_2603_
Setup and press 'Extract all' when prompted

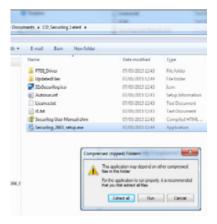

Run through the Extraction wizard that opens clicking 'Extract, Next and Finish'

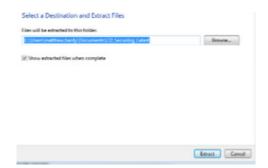

continue over




© Isopharm 2015 www.isopharm.co.uk

Securilog Update Instruction Guide



This should then open a box which looks identical to the box before, again double click on the file called 'Securilog 2603 Setup'

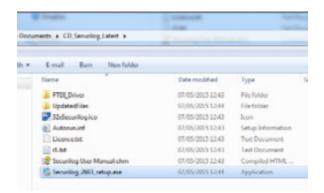

SecuriLOG

should show.

Version 2.6.0.3

Loading file data . . .

This should then install the software and show

Import your files as normal and the latest data

you the usual screen that you are used to.

You should then see the Securilog Software Setup, run through this clicking 'Next, I agree etc'



Please note that the Icon has now changed on your Desktop. The old Golden Triangle has been replaced with a blue icon as below, this will open the exact same program and will not delete any old data.



Isopharm Limited The Decontamination Support Centre, Taylors Court, Rotherham, S62 6NU **T:** +44 (0)1709 52 52 56 **E:** sales@isopharm.co.uk **F:** +44 (0)844 8118 138 **www.isopharm.co.uk**

Registered in England and Wales 3843619

© Isopharm 2015 www.isopharm.co.uk